

Institute of Pathology, Aalborg University Hospital, Ladegaardsgade 3, P.O.Box 561, DK-9100 Aalborg, Denmark

# **Recommended protocol for MSH6**

## Obtained in run 61

30 Dec 2020

Immunostainer	
---------------	--

Type:

Ventana Benchmark Ultra

### Primary antibody

Clone:	SP93
Producer:	Cell Marque
Product no. / lot no.:	287R-24/25/26 / 86841
Diluent:	Antibody Diluent
Dilution factor:	1:100
Incubation time / temperature:	32 min. / 36°C

#### Epitope retrieval, HIER

Device:	On Board / On Machine
Buffer:	Ventana CC1
Heating time at max. temp.:	64 min.
Maximum heating temp.:	98°C

#### **Visualization system**

Producer: Product / no: Incubation time linker: Incubation time polymer: Incubation temperature: Ventana OptiView DAB IHC Detection Kit / 760-700 8 min. 8 min. 36°C

#### Chromogen

Producer:	Ventana
Product / no:	OptiView DAB IHC Detection Kit / 760-700
Incubation time / temperature:	4 min. / 36°C
Enhancement:	CuSO4

#### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.