

## Recommended protocol for MSH6

Obtained in run 52

07 Dec 2017

### Immunostainer

Type: Teecan Freedom Evoware

### Primary antibody

Clone: EP49  
Producer: Immunologic  
Product no. / lot no.: AC-0047 / EN020910  
Diluent: Bright Diluent  
Diluent producer / no.: Immunologic / UD09-125  
Dilution factor: 1:60  
Incubation time / temperature: 30 min. / 22°C

### Epitope retrieval, HIER

Device: PT-link / PT-module  
Buffer: Immunologic HIERbuffer M  
Heating time at max. temp.: 30 min.  
Maximum heating temp.: 99°C

### Visualization system

Producer: Immunologic  
Product / no: Powervision/poly-HRP-GAMs/RbIgG / DPVO-999HRPN  
Linker: None  
Incubation time polymer: 30 min.  
Incubation temperature: 22°C

### Chromogen

Producer: Agilent  
Product / no: DAB / K346811  
Incubation time / temperature: 10 min. / 22°C  
Enhancement: CuSO<sub>4</sub>

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.