

Recommended protocol for p40

Obtained in run 48

17 Aug 2016

Immunostainer

Type: Ventana Benchmark Ultra

Primary antibody

Clone: ZR8
Producer: Immunologic
Product no. / lot no.: ILM0022-C1, -C05 or / Z041406
Diluent: Bright Diluent
Dilution factor: 1:200
Incubation time / temperature: 32 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana Ultra CC1
Heating time at max. temp.: 48 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Amplifier: OptiView Amplification Kit
Incubation time amplifier: 4 min.
Incubation temperature: 36°C

Chromogen

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time / temperature: 8 min. / 36°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.