

## Recommended protocol for p40

Obtained in run 48

17 Aug 2016

### Immunostainer

Type: Ventana Benchmark Ultra

### Primary antibody

Clone: ZR8  
Producer: Immunologic  
Product no. / lot no.: ILM0022-C1, -C05 or / Z041406  
Diluent: Bright Diluent  
Dilution factor: 1:200  
Incubation time / temperature: 32 min. / 36°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Ventana Ultra CC1  
Heating time at max. temp.: 48 min.  
Maximum heating temp.: 100°C

### Visualization system

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time linker: 8 min.  
Incubation time polymer: 8 min.  
Amplifier: OptiView Amplification Kit  
Incubation time amplifier: 4 min.  
Incubation temperature: 36°C

### Chromogen

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time / temperature: 8 min. / 36°C  
Enhancement: CuSO<sub>4</sub>

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.