

Recommended protocol for SYP

Obtained in run 52

24 Jan 2018

Immunostainer

Type: Thermo Autostainer 36/48/72

Primary antibody

Clone: BS15
Producer: Nordic Biosite
Product no. / lot no.: BSH-7385-1 / 001
Diluent: Antibody Diluent
Diluent producer / no.: Nordic Biosite / BCB-20006
Dilution factor: 1:300
Incubation time / temperature: 30 min. / 21°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 20 min.
Maximum heating temp.: 98°C

Visualization system

Producer: Nordic Biosite
Product / no: Anti rabbit HRP polymer / KDB-10046
Linker: Linker, Mouse
Incubation time linker: 10 min.
Incubation time polymer: 30 min.
Incubation temperature: 21°C

Chromogen

Producer: Nordic Biosite
Product / no: High Contrast DAB / BCB-20032
Incubation time / temperature: 10 min. / 21°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.