

## Recommended protocol for SYP

Obtained in run 52

28 Dec 2017

### Immunostainer

Type: Thermo Autostainer 36/48/72

### Primary antibody

Clone: SP11  
Producer: Thermo Fisher Scientific  
Product no. / lot no.: MA5-16402 / 9111S1609J  
Diluent: Primary Antibody Diluent  
Diluent producer / no.: ScyTek / APG999  
Dilution factor: 1:25  
Incubation time / temperature: 60 min. / 25°C

### Epitope retrieval, HIER

Device: PT-link / PT-module  
Buffer: Std Tris-EDTA / EGTA pH 9  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 99°C

### Visualization system

Producer: Immunologic  
Product / no: Brightvision / DPVO999HRP  
Linker: None  
Incubation time polymer: 30 min.  
Incubation temperature: 25°C

### Chromogen

Producer: Immunologic  
Product / no: Bright Dab / BS04-110A  
Incubation time / temperature: 8 min. / 25°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.