

## Recommended protocol for MSH6

Obtained in run 52

29 Dec 2017

### Immunostainer

Type: Ventana Benchmark XT

### Primary antibody

Clone: EPR3945  
Producer: Abcam  
Product no. / lot no.: ab92471 / GR262215-8  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / S2022  
Dilution factor: 1:200  
Incubation time / temperature: 56 min. / 36°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Ventana CC1  
Heating time at max. temp.: 92 min.  
Maximum heating temp.: 99°C

### Visualization system

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time linker: 12 min.  
Incubation time polymer: 12 min.  
Incubation temperature: 37°C

### Chromogen

Producer: Ventana  
Product / no: OptiView DAB IHC Detection Kit / 760-700  
Incubation time / temperature: 8 min. / 37°C  
Enhancement: CuSO<sub>4</sub>

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.