

Recommended protocol for PMS2

Obtained in run 53

08 Feb 2018

Immunostainer

Type: Teecan Freedom Evo

Primary antibody

Clone: EP51
Producer: Immunologic
Product no. / lot no.: AC-0049 / EN051809
Diluent: Bright Diluent
Dilution factor: 1:10
Incubation time / temperature: 30 min. / 22°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Immunologic HIER M Buffer pH8.0
Heating time at max. temp.: 30 min.
Maximum heating temp.: 99°C

Visualization system

Producer: Immunologic
Product / no: Bright VisionpolyHRP-antiMs/Rb/IgG / DPVO-999HRPN
Linker: None
Incubation time polymer: 30 min.
Incubation temperature: 22°C

Chromogen

Producer: DAKO
Product / no: DAB / K3468111
Incubation time / temperature: 10 min. / 22°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.