

Recommended protocol for BSAP

Obtained in run 53

13 Feb 2018

Immunostainer

Type: Teecan Freedom Evo

Primary antibody

Clone: MX017
Producer: Immunologic
Product no. / lot no.: ILM0706-C1 / 160505706
Diluent: Bright Diluent
Diluent producer / no.: Immunologic / UD09-125
Dilution factor: 1:60
Incubation time / temperature: 30 min. / 22°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Klinipath Citrate Buffer pH6.7
Heating time at max. temp.: 30 min.
Maximum heating temp.: 99°C

Visualization system

Producer: Immunologic
Product / no: Bright Vision polyHRP-antiMs/Rb/IgG / DPVO-999HRPN
Linker: None
Incubation time polymer: 30 min.
Incubation temperature: 22°C

Chromogen

Producer: DAKO
Product / no: DAB / K3468111
Incubation time / temperature: 5 min. / 22°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.