

## Recommended protocol for BSAP

Obtained in run 53

13 Feb 2018

### Immunostainer

Type: Teecan Freedom Evo

### Primary antibody

Clone: MX017  
Producer: Immunologic  
Product no. / lot no.: ILM0706-C1 / 160505706  
Diluent: Bright Diluent  
Dilution factor: 1:60  
Incubation time / temperature: 30 min. / 22°C

### Epitope retrieval, HIER

Device: PT-link / PT-module  
Buffer: Klinipath Citrate Buffer pH6.7  
Heating time at max. temp.: 30 min.  
Maximum heating temp.: 99°C

### Visualization system

Producer: Immunologic  
Product / no: Bright Vision polyHRP-antiMs/Rb/IgG / DPVO-999HRPN  
Linker: None  
Incubation time polymer: 30 min.  
Incubation temperature: 22°C

### Chromogen

Producer: DAKO  
Product / no: DAB / K3468111  
Incubation time / temperature: 5 min. / 22°C  
Enhancement: CuSO<sub>4</sub>

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.