

## Recommended protocol for ECAD

Obtained in run 53

23 Feb 2018

### Immunostainer

Type: Ventana Benchmark Ultra

### Primary antibody

Clone: 36B5  
Producer: Leicabiosystems  
Product no. / lot no.: NCL-L-E-Cad / 6054620  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / K8006  
Dilution factor: 1:30  
Incubation time / temperature: 32 min. / 36°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Ventana CC1  
Heating time at max. temp.: 64 min.  
Maximum heating temp.: 100°C

### Visualization system

Producer: Ventana  
Product / no: UltraView Universal DAB Detection Kit / 760-500  
Incubation time polymer: 8 min.  
Amplifier: Ultraview/iView Amplification Kit  
Incubation time amplifier: 8 min.  
Incubation temperature: 36°C

### Chromogen

Producer: Ventana  
Product / no: ultraView Universal DAB Detection Kit / 760-500  
Incubation time / temperature: 8 min. / 36°C  
Enhancement: CuSO<sub>4</sub>

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.