

Recommended protocol for ECAD

Obtained in run 53

01 Mar 2018

Immunostainer

Type: Ventana Benchmark Ultra

Primary antibody

Clone: ECH-6
Producer: Zytomed Systems
Product no. / lot no.: MSK033-05 / S246
Diluent: Antik rper Verd nnungspuffer DCS Lab Line
Diluent producer / no.: DCS / AL120R500
Dilution factor: 1:600
Incubation time / temperature: 60 min. / 37 C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana CC1
Heating time at max. temp.: 64 min.
Maximum heating temp.: 100 C

Visualization system

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time linker: 12 min.
Incubation time polymer: 12 min.
Amplifier: OptiView Amplification Kit
Incubation time amplifier: 12 min.
Incubation temperature: 0 C

Chromogen

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time / temperature: 0 min. / 0 C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect



Nordic Immunohistochemical Quality Control

Institute of Pathology, Aalborg University Hospital, Ladegaardsgade 3, P.O.Box 561, DK-9100 Aalborg, Denmark

the most recent developments in immunohistochemistry at any point in time.