

Recommended protocol for OCT3/4

Obtained in run 53

02 Mar 2018

Immunostainer

Type: Biocare medical Intellipath

Primary antibody

Clone: N1NK
Producer: Leicabiosystems
Product no. / lot no.: NCL-L-OCT3/4 / 6050128
Diluent: Davinci green
Diluent producer / no.: Biocare medical / PD900M
Dilution factor: 1:50
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: Pressure Cooker
Buffer: Biocare medical Diva
Heating time at max. temp.: 20 min.
Maximum heating temp.: 110°C

Visualization system

Producer: Biocare medical
Product / no: Mach4 Universal HRP-polymer / BRI4012L
Linker: None
Incubation time polymer: 10 min.
Incubation temperature: 20°C

Chromogen

Producer: Biocare medical
Product / no: Mach4 universal DAB / BRI4012L
Incubation time / temperature: 5 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.