

## Recommended protocol for OCT3/4

Obtained in run 53

05 Mar 2018

### Immunostainer

Type: Dako Autostainer Link 48 +

### Primary antibody

Clone: MRQ-10  
Producer: Cell Marque  
Product no. / lot no.: 309M-14 / 1612408A  
Diluent: Antibody Diluent  
Diluent producer / no.: Dako / S2022  
Dilution factor: 1:50  
Incubation time / temperature: 30 min. / 22°C

### Epitope retrieval, HIER

Device: PT-link / PT-module  
Buffer: Dako TRS High pH (3-1)  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 97°C

### Visualization system

Producer: Dako  
Product / no: EnVision FLEX+ / K8002/SM802  
Linker: None  
Incubation time polymer: 20 min.  
Incubation temperature: 22°C

### Chromogen

Producer: Dako  
Product / no: DAB+ / DM827 / DM823 / K800  
Incubation time / temperature: 10 min. / 22°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.