

## Recommended protocol for PMS2

Obtained in run 53

07 Mar 2018

### Immunostainer

Type: Manual

### Primary antibody

Clone: A16-4  
Producer: BD Pharmigen  
Product no. / lot no.: 556415 / 7103847  
Diluent: Antibody Dilution Buffer  
Dilution factor: 1:200  
Incubation time / temperature: 30 min. / 20°C

### Epitope retrieval, HIER

Device: Pressure Cooker  
Buffer: Dako TRS High pH (3-1)  
Heating time at max. temp.: 1 min.  
Maximum heating temp.: 120°C

### Visualization system

Producer: Dako/Agilent  
Product / no: FLEX 30+ / K8023  
Linker: Envision FLEX+ Mouse (Linker)  
Incubation time linker: 15 min.  
Incubation time polymer: 30 min.  
Incubation temperature: 20°C

### Chromogen

Producer: Dako  
Product / no: DAB+ / DM827 / DM823 / K800  
Incubation time / temperature: 10 min. / 20°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.