

Recommended protocol for BSAP

Obtained in run 53

07 May 2018

Immunostainer

Type: Thermo Autostainer 36/48/72

Primary antibody

Clone: BSR59
Producer: Nordic Biosite
Product no. / lot no.: BSH-4007-1 / BSH1-1CP
Diluent: Antibody Diluent
Dilution factor: 1:200
Incubation time / temperature: 30 min. / 21°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 20 min.
Maximum heating temp.: 30°C

Visualization system

Producer: Nordic Biosite
Product / no: anti-rabbit HRP polymer / KDB-10046
Linker: None
Incubation time polymer: 30 min.
Incubation temperature: 21°C

Chromogen

Producer: Nordic Biosite
Product / no: High contrast DAB / BCB-20032
Incubation time / temperature: 10 min. / 21°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.