

## Recommended protocol for BSAP

Obtained in run 53

07 May 2018

### Immunostainer

Type: Thermo Autostainer 36/48/72

### Primary antibody

Clone: BSR59  
Producer: Nordic Biosite  
Product no. / lot no.: BSH-4007-1 / BSH1-1CP  
Diluent: Antibody Diluent  
Diluent producer / no.: Nordic Biosite / BCB-20006  
Dilution factor: 1:200  
Incubation time / temperature: 30 min. / 21°C

### Epitope retrieval, HIER

Device: PT-link / PT-module  
Buffer: Std Tris-EDTA / EGTA pH 9  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 30°C

### Visualization system

Producer: Nordic Biosite  
Product / no: anti-rabbit HRP polymer / KDB-10046  
Linker: None  
Incubation time polymer: 30 min.  
Incubation temperature: 21°C

### Chromogen

Producer: Nordic Biosite  
Product / no: High contrast DAB / BCB-20032  
Incubation time / temperature: 10 min. / 21°C  
Enhancement: CuSO<sub>4</sub>

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.