

## Recommended protocol for ECAD

Obtained in run 53

09 Mar 2018

### Immunostainer

Type: Tissue-Tek Genie - Sakura

### Primary antibody

Clone: GM016  
Producer: Tissue-Tek Genie - Sakura Fine  
Product no. / lot no.: 8229-C010 / 8229C01818  
Diluent: None  
Diluent producer / no.: None / None  
Dilution factor: 1:0  
Incubation time / temperature: 30 min. / 25°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Tissue-Tek Genie - S Tissue-Tek Genie High pH Antigen F  
Heating time at max. temp.: 45 min.  
Maximum heating temp.: 98°C

### Visualization system

Producer: Tissue-Tek Genie Sak  
Product / no: Tissue-Tek Genie - Pro Detection Kit, DAB, HRP conjugat  
Linker: Tissue-Tek Genie Link  
Incubation time linker: 15 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 25°C

### Chromogen

Producer: Tissue-Tek Genie Sak  
Product / no: Tissue-Tek Genie - Pro Detection Kit, DAB / 8826-K250  
Incubation time / temperature: 6 min. / 25°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.