

Recommended protocol for ECAD

Obtained in run 53

09 Mar 2018

Immunostainer

Type: Tissue-Tek Genie - Sakura

Primary antibody

Clone: GM016
Producer: Tissue-Tek Genie - Sakura Fine
Product no. / lot no.: 8229-C010 / 8229C01818
Diluent: None
Diluent producer / no.: None / None
Dilution factor: 1:0
Incubation time / temperature: 30 min. / 25°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Tissue-Tek Genie - S Tissue-Tek Genie High pH Antigen F
Heating time at max. temp.: 45 min.
Maximum heating temp.: 98°C

Visualization system

Producer: Tissue-Tek Genie Sak
Product / no: Tissue-Tek Genie - Pro Detection Kit, DAB, HRP conjugat
Linker: Tissue-Tek Genie Link
Incubation time linker: 15 min.
Incubation time polymer: 20 min.
Incubation temperature: 25°C

Chromogen

Producer: Tissue-Tek Genie Sak
Product / no: Tissue-Tek Genie - Pro Detection Kit, DAB / 8826-K250
Incubation time / temperature: 6 min. / 25°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.