

## Recommended protocol for MLA

Obtained in run

15 Sep 2015

### Immunostainer

Type: Ventana Benchmark Ultra

### Primary antibody

Clone: M2-7C10 + M2-9E3  
Producer: BioCare  
Product no. / lot no.: CM077C / 020314  
Diluent: Antibody Diluent with Casein  
Diluent producer / no.: Ventana / 760-219  
Dilution factor: 1:50  
Incubation time / temperature: 40 min. / 37°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Ventana Ultra CC1  
Heating time at max. temp.: 36 min.  
Maximum heating temp.: 95°C

### Visualization system

Producer: Ventana  
Product / no: UltraView Universal Alkaline Phosphatase Red Detection Kit  
Incubation time polymer: 8 min.  
Amplifier: Ultraview/iView Amplification Kit  
Incubation time amplifier: 8 min.  
Incubation temperature: 36°C

### Chromogen

Producer: Ventana  
Product / no: ultraView Universal Alkaline Phosphatase Red Detection Kit  
Incubation time / temperature: 8 min. / 36°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.