

## Recommended protocol for IgL

Obtained in run

21 Aug 2016

### Immunostainer

Type: Dako Autostainer

### Primary antibody

Clone: Polyclonal  
Producer: Dako  
Product no. / lot no.: A0193 / A019302124601  
Diluent: Antibody Diluent  
Dilution factor: 1:5000  
Incubation time / temperature: 30 min. / 20°C

### Epitope retrieval, HIER

Device: Microwave oven  
Buffer: Dako TRS Low pH  
Heating time at max. temp.: 15 min.  
Maximum heating temp.: 100°C

### Visualization system

Producer: Dako  
Product / no: EnVision+, Rabbit / K4002/K4003  
Linker: None  
Incubation time polymer: 30 min.  
Incubation temperature: 20°C

### Chromogen

Producer: Vector  
Product / no: NovaRed / SK-4800  
Incubation time / temperature: 10 min. / 20°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.