

## Recommended protocol for CEA

Obtained in run 54

27 Aug 2018

### Immunostainer

Type: DBS Montage 360 system

### Primary antibody

Clone: COL-1  
Producer: Diagnostic BioSystems  
Product no. / lot no.: MOB008 / D596  
Diluent: Primary Antibody Diluent  
Diluent producer / no.: DBS / K004  
Dilution factor: 1:100  
Incubation time / temperature: 30 min. / 20°C

### Epitope retrieval, HIER

Device: Pressure Cooker  
Buffer: DBS DBS Montage Citrate Antigen Retrieval Solution  
Heating time at max. temp.: 15 min.  
Maximum heating temp.: 105°C

### Visualization system

Producer: DBS  
Product / no: PolyVue PlusMouse/Rabbit HRP / PVP100-AUTO  
Linker: PolyVue Plus Mouse/Rabbit Enhancer  
Incubation time linker: 10 min.  
Incubation time polymer: 10 min.  
Incubation temperature: 20°C

### Chromogen

Producer: DBS  
Product / no: Stable DAB/Plus / K047  
Incubation time / temperature: 5 min. / 20°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.