

Recommended protocol for p63

Obtained in run 48

23 Aug 2016

Immunostainer

Type: DBS Montage 360 system

Primary antibody

Clone: DBR16.1
Producer: Diagnostic BioSystems
Product no. / lot no.: RMPD086R / F175-NB
Diluent: Primary Antibody Diluent
Dilution factor: 1:100
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: Pressure Cooker
Buffer: DBS Montage EDTA Antigen Retrieval Solution
Heating time at max. temp.: 20 min.
Maximum heating temp.: 105°C

Visualization system

Producer: Diagnostic BioSystem
Product / no: PolyVue PlusMouse/Rabbit HRP / PVP 100D
Linker: Mouse and Rabbit Link
Incubation time linker: 10 min.
Incubation time polymer: 10 min.
Incubation temperature: 20°C

Chromogen

Producer: Diagnostic BioSystem
Product / no: Stable DAB/Plus / K047
Incubation time / temperature: 5 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.