

Recommended protocol for PR

Obtained in run B26

28 Aug 2018

Immunostainer

Type: Dako Autostainer Link 48 +

Primary antibody

Clone: 16
Producer: Leica/novocastra
Product no. / lot no.: NCL-L-PR-312 / 6052038
Diluent: Antibody Diluent
Dilution factor: 1:200
Incubation time / temperature: 30 min. / 23°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Dako TRS High pH (3-1)
Heating time at max. temp.: 20 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako
Product / no: EnVision FLEX+ / K8002/SM802
Linker: None
Incubation time polymer: 20 min.
Incubation temperature: 23°C

Chromogen

Producer: Dako
Product / no: DAB+ / DM827 / DM823 / K800
Incubation time / temperature: 10 min. / 23°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.