

## Recommended protocol for SOX10

Obtained in run 55

07 Dec 2018

### Immunostainer

Type: Dako Omnis

### Primary antibody

Clone: BC34  
Producer: Biocare Medical  
Product no. / lot no.: AC13099A or C / 070318  
Diluent: Antibody Diluent  
Dilution factor: 1:25  
Incubation time / temperature: 20 min. / 32°C

### Epitope retrieval, HIER

Device: PT-link / PT-module  
Buffer: Dako Omnis Target Retrieval Solution, High pH  
Heating time at max. temp.: 30 min.  
Maximum heating temp.: 97°C

### Visualization system

Producer: Dako Omnis  
Product / no: EnVision Flex / GV800/GV823  
Linker: Mouse LINKER  
Incubation time linker: 10 min.  
Incubation time polymer: 20 min.  
Incubation temperature: 32°C

### Chromogen

Producer: Dako Omnis  
Product / no: DAB+ Substrate Chromogen System / GV825  
Incubation time / temperature: 5 min. / 32°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.