

Recommended protocol for DES

Obtained in run 48

29 Aug 2016

Immunostainer

Type: IntelliPath Biocare Medical

Primary antibody

Clone: D33
Producer: Dako
Product no. / lot no.: M0760 / 20013834
Diluent: Da vinci green
Dilution factor: 1:50
Incubation time / temperature: 30 min. / 20°C

Epitope retrieval, HIER

Device: Pressure Cooker
Buffer: Biocare Medical Diva
Heating time at max. temp.: 15 min.
Maximum heating temp.: 110°C

Visualization system

Producer: Biocare Medical
Product / no: Mach4 Univ.HRP-polymer / BRI4012L
Linker: None
Incubation time polymer: 10 min.
Incubation temperature: 20°C

Chromogen

Producer: Biocare Medical
Product / no: DAB Mach4 universal / BRI4012L
Incubation time / temperature: 5 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.