

## Recommended protocol for C-MYC

Obtained in run 56

05 Feb 2019

### Immunostainer

Type: Leica BOND III

### Primary antibody

Clone: Ep121  
Producer: Cell Marque  
Product no. / lot no.: 395R-16 / 0000030174  
Diluent: Bond Antibody Diluent  
Dilution factor: 1:100  
Incubation time / temperature: 60 min. / 22°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Leica Bond Epitope Retrieval Solution 2  
Heating time at max. temp.: 40 min.  
Maximum heating temp.: 99°C

### Visualization system

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time linker: 15 min.  
Incubation time polymer: 15 min.  
Incubation temperature: 22°C

### Chromogen

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time / temperature: 10 min. / 22°C  
Enhancement: CuSO<sub>4</sub>

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.