

Recommended protocol for CD117

Obtained in run 56

01 Mar 2019

Immunostainer

Type: Manual

Primary antibody

Clone: MX041
Producer: maixin
Product no. / lot no.: MAB-0745 / 190220745a
Diluent: None
Dilution factor: 1:0
Incubation time / temperature: 60 min. / 25°C

Epitope retrieval, HIER

Device: Water bath
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 20 min.
Maximum heating temp.: 100°C

Visualization system

Producer: maixin
Product / no: MaxVision III Ultra DAB / Kit-0038
Linker: None
Incubation time polymer: 30 min.
Incubation temperature: 25°C

Chromogen

Producer: maixin
Product / no: MaxVision III Ultra DAB / Kit-0038
Incubation time / temperature: 5 min. / 25°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.