

## Recommended protocol for MLA

Obtained in run 56

07 Mar 2019

### Immunostainer

Type: Manual

### Primary antibody

Clone: M2-7C10  
Producer: Zytomed Systems  
Product no. / lot no.: MSK056 / W783  
Diluent: Antibody Diluent  
Diluent producer / no.: Zytomed Systems / ZUC025  
Dilution factor: 1:100  
Incubation time / temperature: 45 min. / 22°C

### Epitope retrieval, HIER

Device: Pressure Cooker  
Buffer: Zytomed-Systems HIER-Citrate  
Heating time at max. temp.: 7 min.  
Maximum heating temp.: 110°C

### Visualization system

Producer: Zytomed-Systems  
Product / no: HRP-Polymer / POLHRP-100  
Linker: Postblock  
Incubation time linker: 20 min.  
Incubation time polymer: 30 min.  
Incubation temperature: 22°C

### Chromogen

Producer: Zytomed Systems  
Product / no: DAB high contrast / DAB500plus  
Incubation time / temperature: 10 min. / 22°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.