

Recommended protocol for SOX10

Obtained in run 48

31 Aug 2016

Immunostainer

Type: Biocare IntelliPath

Primary antibody

Clone: BC34
Producer: Biocare Medical
Product no. / lot no.: AC13099A or C / 050515
Diluent: Bio Site Histo Antibody Diluent
Dilution factor: 1:100
Incubation time / temperature: 30 min. / 23°C

Epitope retrieval, HIER

Device: Pressure Cooker
Buffer: Biocare Diva
Heating time at max. temp.: 5 min.
Maximum heating temp.: 110°C

Visualization system

Producer: Biocare
Product / no: Mach 4 Universal HRP-Polymer Detection System / BRI40
Incubation time polymer: 10 min.
Incubation temperature: 23°C

Chromogen

Producer: as per kit
Product / no: DAB as per kit / as per kit
Incubation time / temperature: 5 min. / 23°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.