

## Recommended protocol for SOX10

Obtained in run 48

31 Aug 2016

### Immunostainer

Type: Biocare IntelliPath

### Primary antibody

Clone: BC34  
Producer: Biocare Medical  
Product no. / lot no.: AC13099A or C / 050515  
Diluent: Bio Site Histo Antibody Diluent  
Diluent producer / no.: Bio Site / BCB20006  
Dilution factor: 1:100  
Incubation time / temperature: 30 min. / 23°C

### Epitope retrieval, HIER

Device: Pressure Cooker  
Buffer: Biocare Diva  
Heating time at max. temp.: 5 min.  
Maximum heating temp.: 110°C

### Visualization system

Producer: Biocare  
Product / no: Mach 4 Universal HRP-Polymer Detection System / BRI40  
Incubation time polymer: 10 min.  
Incubation temperature: 23°C

### Chromogen

Producer: as per kit  
Product / no: DAB as per kit / as per kit  
Incubation time / temperature: 5 min. / 23°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.