

Recommended protocol for PD-L1 (TPS/CPS)

Obtained in run C6

23 Aug 2019

Immunostainer

Type: Dako Autostainer Link 48 +

Primary antibody

Clone: 28-8
Producer: Dako
Product no. / lot no.: SK005 / 10148177
Format: Ready-To-Use (prediluted)
Incubation time / temperature: 30 min. / 23°C

Epitope retrieval, HIER

Device: PT-link / PT-module
Buffer: Dako TRS Low pH (3-1)
Heating time at max. temp.: 20 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako
Product / no: PD-L1 IHC 28-8 pharmDx / SK005
Incubation time linker: 30 min.
Incubation time polymer: 30 min.
Incubation temperature: 23°C

Chromogen

Producer: Dako
Product / no: PD-L1 IHC 28-8 pharmDx / SK005
Incubation time / temperature: 10 min. / 23°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.