

Recommended protocol for PD-L1 (IC)

Obtained in run C6

26 Aug 2019

Immunostainer

Type: Ventana Benchmark Ultra

Primary antibody

Clone: CAL10
Producer: Biocare Medical
Product no. / lot no.: ACI3171C / 080317
Diluent: Renaissance Background Reducing Diluent
Diluent producer / no.: Biocare Medical / PD905L
Dilution factor: 1:50
Incubation time / temperature: 32 min. / 36°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Ventana Ultra CC1
Heating time at max. temp.: 64 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Amplifier: OptiView Amplification Kit
Incubation time amplifier: 4 min.
Incubation temperature: 36°C

Chromogen

Producer: Ventana
Product / no: OptiView DAB IHC Detection Kit / 760-700
Incubation time / temperature: 8 min. / 36°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect



Nordic Immunohistochemical Quality Control

Institute of Pathology, Aalborg University Hospital, Ladegaardsgade 3, P.O.Box 561, DK-9100 Aalborg, Denmark

the most recent developments in immunohistochemistry at any point in time.