

## Recommended protocol for HER2 IHC

Obtained in run B28

27 Aug 2019

### Immunostainer

Type: Leica BOND Max

### Primary antibody

Clone: SP3  
Producer: Thermo Scientific  
Product no. / lot no.: RM-9103-S-A / 9103S1807B  
Diluent: Bond Antibody Diluent  
Diluent producer / no.: Leica / AR9352  
Dilution factor: 1:100  
Incubation time / temperature: 30 min. / 20°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Leica Bond Epitope Retrieval Solution 2  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 99°C

### Visualization system

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time linker: 8 min.  
Incubation time polymer: 8 min.  
Incubation temperature: 20°C

### Chromogen

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time / temperature: 10 min. / 20°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.