

Recommended protocol for p63

Obtained in run 48

05 Sep 2016

Immunostainer

Type: Thermo Autostainer 36/48/72

Primary antibody

Clone: DAK-p63
Producer: Dako
Product no. / lot no.: M7317 / 20017563
Diluent: Antibody Diluent
Dilution factor: 1:50
Incubation time / temperature: 30 min. / 22°C

Epitope retrieval, HIER

Device: Other
Buffer: home brew TRIS EDTA pH 9,0
Heating time at max. temp.: 10 min.
Maximum heating temp.: 100°C

Visualization system

Producer: Dako
Product / no: REAL EnVision Rabbit/Mouse / K5007 / K4065
Incubation time polymer: 30 min.
Incubation temperature: 22°C

Chromogen

Producer: Dako
Product / no: DAB+ / K5007 / K4065
Incubation time / temperature: 15 min. / 22°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.