

Recommended protocol for NKX3.1

Obtained in run 58

10 Dec 2019

Immunostainer

Type: Dako Omnis

Primary antibody

Clone: Polyclonal
Producer: Biocare Medical
Product no. / lot no.: CP 422 / 050819
Diluent: Renoir Red Diluent
Dilution factor: 1:50
Incubation time / temperature: 30 min. / 32°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Dako Omnis Target Retrieval Solution, High pH
Heating time at max. temp.: 24 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Dako Omnis
Product / no: EnVision Flex / GV800/GV823
Linker: Rabbit LINKER
Incubation time linker: 10 min.
Incubation time polymer: 20 min.
Incubation temperature: 32°C

Chromogen

Producer: Dako Omnis
Product / no: DAB+ Substrate Chromogen System / GV825
Incubation time / temperature: 10 min. / 32°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.