

Recommended protocol for HER2 IHC

Obtained in run B29

11 Dec 2019

Immunostainer

Type: Leica BOND III

Primary antibody

Clone: Oracle (clone CB11)
Producer: Leica/novocastra
Product no. / lot no.: TA9145 / 64348
Format: Ready-To-Use (prediluted)
Incubation time / temperature: 30 min. / 22°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Leica Bond Epitope Retrieval Solution 1
Heating time at max. temp.: 25 min.
Maximum heating temp.: 97°C

Visualization system

Producer: Leica
Product / no: Oracle / TA9145
Incubation time linker: 10 min.
Incubation time polymer: 10 min.
Incubation temperature: 22°C

Chromogen

Producer: Leica
Product / no: Oracle / TA9145
Incubation time / temperature: 10 min. / 22°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.