

## Recommended protocol for SATB2

Obtained in run 58

12 Dec 2019

### Immunostainer

Type: Leica BOND III

### Primary antibody

Clone: OTI5H7  
Producer: ZHONGSHANJINQIAO  
Product no. / lot no.: ZM-0163 / 19073007  
Diluent: Bond Antibody Diluent  
Dilution factor: 1:100  
Incubation time / temperature: 30 min. / 20°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Leica Bond Epitope Retrieval Solution 2  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 95°C

### Visualization system

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time linker: 8 min.  
Incubation time polymer: 8 min.  
Incubation temperature: 20°C

### Chromogen

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time / temperature: 10 min. / 20°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.