

Recommended protocol for p40

Obtained in run

22 Sep 2015

Immunostainer

Type: Autostainer, LabVision

Primary antibody

Clone: ZR8
Producer: Immunologic
Product no. / lot no.: ILM0022-C1 / Z041406
Diluent: Antibody Diluent
Dilution factor: 1:800
Incubation time / temperature: 60 min. / 20°C

Epitope retrieval, HIER

Device: Pressure Cooker
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 10 min.
Maximum heating temp.: 126°C

Visualization system

Producer: Immunologic
Product / no: NS / DPVB-999HRP
Linker: NS
Incubation time linker: 15 min.
Incubation time polymer: 30 min.
Incubation temperature: 20°C

Chromogen

Producer: Immunologic
Product / no: DAB / BS04-999
Incubation time / temperature: 8 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.