

## Recommended protocol for PR

Obtained in run

22 Sep 2015

### Immunostainer

Type: Xmatrx Elite, Biogenex

### Primary antibody

Clone: EP2  
Producer: BioGenex  
Product no. / lot no.: AN711-5M / AN7111013  
Diluent: Antibody Diluent  
Dilution factor: 1:50  
Incubation time / temperature: 30 min. / 20°C

### Epitope retrieval, HIER

Device: Microwave oven  
Buffer: Std Citrate pH 6  
Heating time at max. temp.: 10 min.  
Maximum heating temp.: 100°C

### Visualization system

Producer: BioGenex  
Product / no: NS / QD400-60KE  
Linker: NS  
Incubation time linker: 20 min.  
Incubation time polymer: 30 min.  
Incubation temperature: 25°C

### Chromogen

Producer: BioGenex  
Product / no: DAB / HK153-5K  
Incubation time / temperature: 10 min. / 25°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.