

Recommended protocol for MLA

Obtained in run

22 Sep 2015

Immunostainer

Type: Manual

Primary antibody

Clone: M2-7C10
Producer: Zytomed Systems
Product no. / lot no.: MSK056 / L518
Diluent: Antibody Diluent
Dilution factor: 1:100
Incubation time / temperature: 60 min. / 20°C

Epitope retrieval, HIER

Device: Pressure Cooker
Buffer: Std Citrate pH 6
Heating time at max. temp.: 10 min.
Maximum heating temp.: 110°C

Visualization system

Producer: Zytomed Systems
Product / no: NS / POLHRP-100
Linker: NS
Incubation time linker: 20 min.
Incubation time polymer: 30 min.
Incubation temperature: 20°C

Chromogen

Producer: Zytomed Systems
Product / no: DAB+ / DAB500+
Incubation time / temperature: 10 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.