

Recommended protocol for Bcl-6

Obtained in run

23 Sep 2015

Immunostainer

Type: Autostainer, LabVision

Primary antibody

Clone: PG-B6p
Producer: Dako
Product no. / lot no.: M7211 / 00095021
Diluent: Antibody Diluent
Dilution factor: 1:50
Incubation time / temperature: 60 min. / 20°C

Epitope retrieval, HIER

Device: Pressure Cooker
Buffer: Std Tris-EDTA / EGTA pH 9
Heating time at max. temp.: 10 min.
Maximum heating temp.: 126°C

Visualization system

Producer: Immunnologic
Product / no: NS / DPVB-999HRP
Linker: NS
Incubation time linker: 15 min.
Incubation time polymer: 30 min.
Incubation temperature: 20°C

Chromogen

Producer: Immunologic
Product / no: DAB / BS04-999
Incubation time / temperature: 8 min. / 20°C
Enhancement: None

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.