

Recommended protocol for ASMA

Obtained in run 59

03 Feb 2020

Immunostainer

Type: Leica BOND III

Primary antibody

Clone: ?sm-1
Producer: Leicabiosystems
Product no. / lot no.: PA0943 / 65028
Format: Ready-To-Use (prediluted)
Incubation time / temperature: 15 min. / 20°C

Visualization system

Producer: Leica
Product / no: Bond Refine / DS9800
Incubation time linker: 8 min.
Incubation time polymer: 8 min.
Incubation temperature: 20°C

Chromogen

Producer: Leica
Product / no: Bond Refine / DS9800
Incubation time / temperature: 10 min. / 20°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.