

## Recommended protocol for p57

Obtained in run

23 Sep 2015

### Immunostainer

Type: Freedom Evo, Teecan

### Primary antibody

Clone: 57P06 (KP10)  
Producer: Thermo Scientific  
Product no. / lot no.: MS-1062-P / 1062P1310C  
Diluent: Antibody Diluent  
Dilution factor: 1:2000  
Incubation time / temperature: 30 min. / 20°C

### Epitope retrieval, HIER

Device: PT-link / PT-module  
Buffer: Std Citrate pH 6  
Heating time at max. temp.: 30 min.  
Maximum heating temp.: 100°C

### Visualization system

Producer: Immunologic  
Product / no: NS / DPVO-999HRPN  
Incubation time polymer: 30 min.  
Incubation temperature: 20°C

### Chromogen

Producer: Dako  
Product / no: DAB+ / K3467 / K3468  
Incubation time / temperature: 5 min. / 20°C  
Enhancement: CuSO<sub>4</sub>

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.