

## Recommended protocol for WT1

Obtained in run

23 Sep 2015

### Immunostainer

Type: IntelliPath, Biocare

### Primary antibody

Clone: 6F-H2  
Producer: Dako  
Product no. / lot no.: M3561 / 0093F  
Diluent: Antibody Diluent  
Dilution factor: 1:200  
Incubation time / temperature: 30 min. / 20°C

### Epitope retrieval, HIER

Device: Pressure Cooker  
Buffer: Std Tris-EDTA / EGTA pH 9  
Heating time at max. temp.: 5 min.  
Maximum heating temp.: 103°C

### Visualization system

Producer: Immunologic  
Product / no: NS / DPVB-110HRP  
Linker: NS  
Incubation time linker: 15 min.  
Incubation time polymer: 25 min.  
Incubation temperature: 20°C

### Chromogen

Producer: Dako  
Product / no: DAB+ / K3467 / K3468  
Incubation time / temperature: 5 min. / 20°C  
Enhancement: CuSO<sub>4</sub>

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.