

## Recommended protocol for ASMA

Obtained in run 59

05 Mar 2020

### Immunostainer

Type: Leica BOND III

### Primary antibody

Clone: 1A4  
Producer: Dako  
Product no. / lot no.: M0851 / 20052340  
Diluent: Bond Antibody Diluent  
Diluent producer / no.: Leica / AR9352  
Dilution factor: 1:300  
Incubation time / temperature: 15 min. / 21°C

### Visualization system

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time linker: 8 min.  
Incubation time polymer: 8 min.  
Incubation temperature: 21°C

### Chromogen

Producer: Leica  
Product / no: Bond Refine / DS9800  
Incubation time / temperature: 10 min. / 21°C  
Enhancement: CuSO<sub>4</sub>

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.