

Recommended protocol for MLA

Obtained in run 60

05 Aug 2020

Immunostainer

Type: Sakura Tissue-Tek Genie

Primary antibody

Clone: EP43
Producer: Genmed/Sakura Finetek
Product no. / lot no.: 8319 / na
Diluent: None
Diluent producer / no.: None / None
Dilution factor: 1:0
Incubation time / temperature: 30 min. / 25°C

Epitope retrieval, HIER

Device: On Board / On Machine
Buffer: Sakura Finetek Sakura-High pH Antigen retrieval buffer
Heating time at max. temp.: 45 min.
Maximum heating temp.: 98°C

Visualization system

Producer: Sakura Finetek
Product / no: Tissue-Tek Genie Pro / 8826-K250
Incubation time polymer: 20 min.
Incubation temperature: 25°C

Chromogen

Producer: Sakura-Finetek
Product / no: DAB / in detection kit 882
Incubation time / temperature: 6 min. / 25°C
Enhancement: CuSO₄

Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.