

## Recommended protocol for MLA

Obtained in run 60

06 Aug 2020

### Immunostainer

Type: Dako Autostainer Link 48 +

### Primary antibody

Clone: Melan-A rabbit monoclonal  
Producer: abcam  
Product no. / lot no.: ab51061 / GR125107-28  
Diluent: Antibody Diluent  
Dilution factor: 1:200  
Incubation time / temperature: 30 min. / 23°C

### Epitope retrieval, HIER

Device: PT-link / PT-module  
Buffer: Dako TRS High pH (3-1)  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 97°C

### Visualization system

Producer: Dako  
Product / no: EnVision FLEX+ / K8002/SM802  
Linker: None  
Incubation time polymer: 20 min.  
Incubation temperature: 23°C

### Chromogen

Producer: Dako  
Product / no: DAB+ / DM827 / DM823 / K800  
Incubation time / temperature: 10 min. / 23°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.