

## Recommended protocol for MLA

Obtained in run 60

10 Aug 2020

### Immunostainer

Type: Ventana Benchmark Ultra

### Primary antibody

Clone: EP43  
Producer: Nordic Biosite  
Product no. / lot no.: AC-0041 / 20040201  
Diluent: Antibody Diluent  
Dilution factor: 1:25  
Incubation time / temperature: 32 min. / 36°C

### Epitope retrieval, HIER

Device: On Board / On Machine  
Buffer: Ventana CC1  
Heating time at max. temp.: 64 min.  
Maximum heating temp.: 95°C

### Visualization system

Producer: Ventana  
Product / no: UltraView Universal Alkaline Phosphatase Red Detection Kit  
Incubation time polymer: 8 min.  
Incubation temperature: 36°C

### Chromogen

Producer: Ventana  
Product / no: ultraView Universal Alkaline Phosphatase Red Detection Kit  
Incubation time / temperature: 8 min. / 36°C  
Enhancement: None

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.