

## Recommended protocol for MLA

Obtained in run 60

19 Oct 2020

### Immunostainer

Type: Thermo Autostainer 36/48/72

### Primary antibody

Clone: BS52  
Producer: Nordic Biosite  
Product no. / lot no.: BSH-2003-1 / BSH1-7ac  
Diluent: Antibody Diluent  
Dilution factor: 1:100  
Incubation time / temperature: 30 min. / 21°C

### Epitope retrieval, HIER

Device: PT-link / PT-module  
Buffer: Std Tris-EDTA / EGTA pH 9  
Heating time at max. temp.: 20 min.  
Maximum heating temp.: 98°C

### Visualization system

Producer: BioSite Histo  
Product / no: (HRP) Polymer anti-Mouse / KDB-OZRNGT-100  
Linker: None  
Incubation time polymer: 30 min.  
Incubation temperature: 21°C

### Chromogen

Producer: BioSite Histo  
Product / no: DAB Substrate Buffer / BCB-R7IKBJ-1  
Incubation time / temperature: 10 min. / 21°C  
Enhancement: CuSO<sub>4</sub>

### Disclaimer:

NordiQC makes every attempt to provide accurate and up-to-date information, yet NordiQC does not make any claim or warranty regarding the accuracy of the provided information nor does it represent that the contents of the web site and protocols reflect the most recent developments in immunohistochemistry at any point in time.